

# seca mBCA 555/seca TRU RFID-Reader assemble

Install your  
RFID reader  
in just a  
few steps

EMR))



# seca mBCA 555/seca TRU



1. Place the RFID reader in the middle the front of the display holder. Attach the RFID reader with the double-sided adhesive strip in this position.



2. Plug the USB connector of the RFID reader on the back of the display into the free USB port and run the cable as depicted by the opening provided for this purpose.



3. After the cable has been completely routed through the opening provided, route the cable in the display holder in the cable guide as shown.



4. Carefully place the cover back on the display holder and run the excess cable in the guide of the aluminum profile. Finally, place the plastic cover on the aluminum profile to cover the cables.