

Dear new customer!

Welcome to the seca family! Use this checklist to ensure that your mBCA is set up completely and correctly and successfully integrated into your network and the seca Cloud. Regardless of whether you set it up yourself or it was done for you. Follow the steps to ensure smooth commissioning.

Step	What needs to be done?	How do I do that?	Done?
1	Assembly of the device	<p>We have prepared some videos for this. Here you can find the instructions with size measuring stick. (at least 2.5 m ceiling height) Here are the instructions without a measuring stick. If you prefer to have the instructions as a document, you can find them here. All devices should be 1 m away from other technical devices.</p>	
2	Enable port	<p>For the data transfer to the cloud, the outgoing port 22020 must be enabled on your router. Protocol: TCP; Connection: HTTPS; Outgoing connection; Destination address is: gpx.secacloud.com Please contact your IT service provider if you cannot do this yourself.</p>	
3	Network connection	<p>If you want to connect your seca mBCA via WLAN, you will find the instructions here.</p> <p>If your seca mBCA is to be connected via network cable, you will find the instructions here. Please note that the network socket must be within 3 m and must be activated. For larger distances, please have an appropriate network cable ready.</p>	
4	Cloud access	<p>The seca Cloud can be accessed via https://www.secacloud.com. You have received the following login details in advance by email:</p> <ul style="list-style-type: none"> - external Tenant ID - Your user name - The email address you use <p>You should have already created your own user. If you have forgotten your password, you can go to Forgotten password and you will then receive a link to create a new password at your e-mail address above.</p>	