

seca mBCA 555/seca TRU

Log in to the seca Cloud

How can I
log in to the
seca cloud
in just a few
steps



seca mBCA 555/seca TRU

1. Start a browser (e.g. Chrome) on an end device and open the following website:
<https://secacloud.com>
We have a log-in page where you can either log in with a business account or access your personal data as a patient or member of a studio. The choice is yours!




2. On the log-in page that opens, you can now either log in with your business account (seca analytics 125) or, if you are a patient or member of a studio, retrieve your personal data (seca myAnalytics). To switch between the two options, click on the account you wish to switch to on the right-hand side of the log-in page. This changes the input mask on the left-hand side.
3. To log in with the business account, you need your External Tenant ID, your user name and your password. The input mask will look like this:

seca analytics 125

Log in to your business account

External Tenant ID

Username

Password 

[Forgot password?](#)

Login

You do not know your External Tenant ID or need help? Then please contact the seca service and create a customer enquiry under the following link: <https://support.seca.com/>

seca mBCA 555/seca TRU

4. As a patient or member of a studio, switch to the seca myAnalytics account and enter your email and password. This will look like this:

seca myAnalytics

Log in to your user account

[Forgot password?](#)

Remember me

Don't have an account? [Sign up](#)