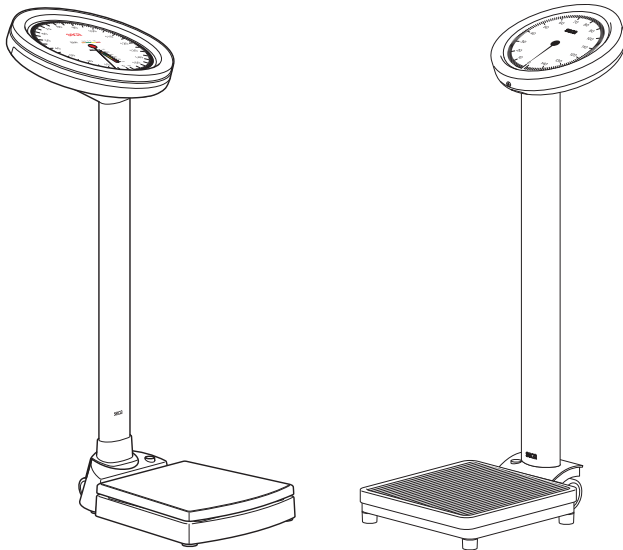
seca 755/ 756/786

Gebrauchsanleitung und Garantieerklärung	3
Instruction manual and guarantee	25
Mode d'emploi et garantie	45
Manuale di istruzioni e garanzia	65
Manual de instrucciones y garantia	85

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CONTENTS

1. Device description	26	4.4 Setting up and aligning the scale	36
1.1 Intended use	26	5. Weighing	37
1.2 Functional description	26	5.1 Adjusting the display	37
1.3 User qualification	26	5.2 Performing weighing	37
1.4 Patient target group	26	5.3 Reading off Body Mass Index (seca 755/756 only)	38
1.5 Contraindications	26	6. Hygienic reprocessing	38
1.6 Clinical benefit	26	6.1 Cleaning	38
2. Safety information	27	6.2 Desinfection	39
2.1 Safety information in this manual	27	6.3 Sterilisation	39
2.2 Basic safety precautions	27	7. Function check	39
Handling the device	27	8. Troubleshooting	40
Avoiding infections	28	9. Maintenance	40
Avoiding injury	28	9.1 Information on servicing (seca 755/786)	40
Avoiding damage to the device	29	9.2 Information on verification (seca 756)	41
Handling the measurement results	29	10. Technical data	41
Handling the packaging material	30	10.1 General technical data	41
3. Overview	31	10.2 Technical data, weighing	42
3.1 Controls/Features	31	11. Optional accessories	43
3.2 Markings on the device and on the type plate	32	12. Disposal	43
3.3 Markings on the packaging	33	13. Warranty	43
4. Starting up the device	34	14. Declaration of conformity	44
4.1 Scope of delivery	34		
4.2 Assembling the device	34		
4.3 Transporting the device	35		

1. DEVICE DESCRIPTION

1.1 Intended use

The mechanical column scale supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight or weight changes.

1.2 Functional description

With a mechanical dial scale weight is calculated via a load lever system.

Before weighing takes place, the zero point can be manually reset each time with the adjusting wheel.

Calibrated scales are supplied with a kg scale.

Uncalibrated scales are supplied with a kg, lbs, kg/lbs or a kg/sts scale depending on the model.

1.3 User qualification

The device may only be operated by persons with formal training in the field of healthcare or medicine.

1.4 Patient target group

Persons of any age who do not exceed the maximum capacity of the scale and can stand independently on the scale.

1.5 Contraindications

No contraindications are known.

1.6 Clinical benefit

The mechanical column scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight. By that, the mechanical column scale only provides an indirect clinical benefit. The physician must perform additional examinations for an accurate diagnosis.

2. SAFETY INFORMATION

2.1 Safety information in this manual

**DANGER!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

**WARNING!**

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

**CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

2.2 Basic safety precautions

Handling the device

- Please take note of the information in these instructions for use.
- Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing and subsequent verification carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any parts for servicing by the user. Only have servicing and repairs performed by an authorized

seca service partner. You can find service partners in your area at www.seca.com or by sending an e-mail to service@seca.com.

- ▶ Only use original seca accessories and spare parts, otherwise seca will not grant any warranty.

Avoiding infections

WARNING!

Risk of infection

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Hygienically reprocess the device regularly as described in the respective section in this document.
- ▶ Make sure that the patient has no infectious diseases.
- ▶ Make sure that the patient has no open wounds or infectious skin alterations, which may come into contact with the device.

Avoiding injury

WARNING!

Injury from falls

- ▶ Ensure that the device is positioned firmly and level.
- ▶ Support people with restricted mobility when they step onto and off the scale.
- ▶ Only use this scale for patients who are able to stand on their own when being weighed.
- ▶ Make sure that the patient does not step onto and off the weighing platform right at the edges.
- ▶ Make sure that the patient steps onto and off the weighing platform slowly and safely.

WARNING!

Danger of slipping

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patients feet are dry before he or she steps onto the weighing platform.
- ▶ Make sure that the patient steps onto and off the weighing platform slowly and safely.

Avoiding damage to the device

NOTICE!

Damage to device

- ▶ Make sure not to drop the device.
- ▶ Do not expose the device to any impact or vibration.
- ▶ Perform function controls regularly as described in the relevant section in this document. Do not operate the device if it is damaged or not working properly.
- ▶ Caustic detergents may damage the surfaces. Only use a soft cloth dampened with mild soapsuds to clean the surfaces of the device.
- ▶ Only use disinfectants suitable for sensitive surfaces. Suitable disinfectants are available from specialist dealers.

Handling the measurement results

CAUTION!

Patient hazard

In order to avoid misinterpretations, measuring results for medical use must be displayed and used in SI units (weight: kilogrammes, length: metres) only. Some devices offer the ability to display measuring results in other units. This is only an additional function.

- ▶ Use the results exclusively in SI units.
- ▶ The use of measuring results in non-SI units is the sole responsibility of the user.

NOTICE!

Inconsistent measuring results

- ▶ Before you electronically save measured values determined using this device and use them further (e.g. in seca software or in an EMR system), make sure that the measured values are plausible.
- ▶ If measured values are transmitted to seca software or an EMR system, make sure prior to further use that the measured values are plausible and are assigned to the correct patient.

Handling the packaging material

WARNING!

Risk of suffocation

Packaging material made of plastic foil (bags) is a choking hazard.

- ▶ Keep packaging material out of reach of children.
- ▶ In the event that the original packing material may not be available anymore, only use plastic bags with security holes in order to reduce the risk of suffocation. Use recyclable materials if possible.

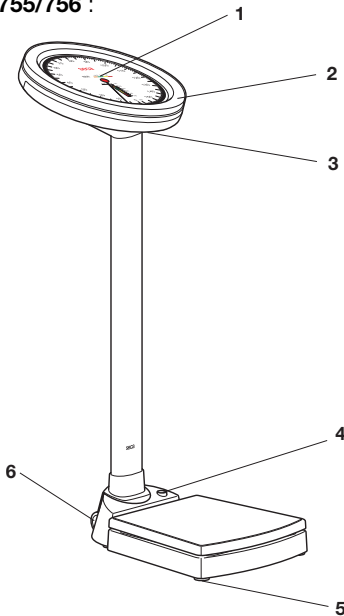
NOTE

Keep the original packing material for future use (e.g. returning for servicing).

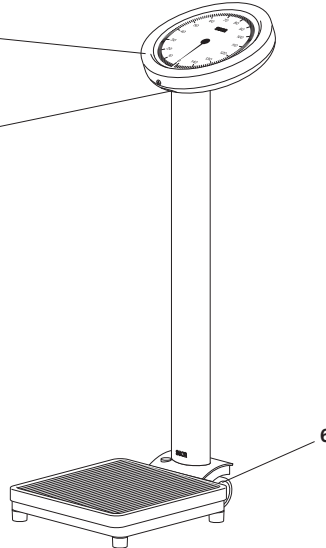
3. OVERVIEW

3.1 Controls/Features

seca 755/756 :

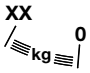
seca 786 :


No.	Control	Function
1	Color scale	Used for reading off the Body Mass Index (only seca 755/756)

No.	Control	Function
2	Scale ring	Protects the pointer and dial
3	Adjusting wheel	For adjusting the display
4	Spirit level	Indicates whether the device is horizontal (verified models)
5	Foot screw	Used for precise alignment of the device (4 pcs)
6	Transport rollers	Used for transporting over short distances (2 pcs)

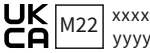3.2 Markings on the device and on the type plate

Text/Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
	Model number
	Serial number
ProdID	Product identification number, consecutive
Approval Type	Type designation of design approval
	Follow instructions for use
e	Value in units of measurement used to classify and calibrate a scale (calibrated models)
d	Value in units of measurement which gives the difference between two consecutive display values (non-verified models)
	Class III scale in compliance with directive 2014/31/EU
	<ul style="list-style-type: none"> • Unit used for scale (here: kg) • Part of scale below the minimum load: For technical reasons no meaningful measured results possible in this range

Text/Symbol	Meaning
	<p>Device complies with EU directives.</p> <ul style="list-style-type: none"> • M: Conformity label in compliance with directive 2014/31/EU for non-automatic weighing instruments (verified models) • 20: (Example: 2020) Year in which conformity verification was performed and the CE label was applied (verified models) • 0102: Appointed office for metrology (verified models) • 0123: Appointed office for medical devices
	<p>Medical device in accordance with Regulation (EU) 2017/745</p>
	<p>Device complies with United Kingdom directives</p> <ul style="list-style-type: none"> • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models) • 22: (Example: 2022) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
	<p>Importer/representative in the United Kingdom: seca Ltd 40 Barn Street Birmingham B5 5QB United Kingdom</p>
	<p>Importer/representative in Switzerland: seca ag (schweiz) Medical scales and measuring systems Schönmatt Str. 2 CH-4153 REINACH</p>


3.3 Markings on the packaging

Text/symbol	Meaning
	<p>Protect from moisture</p>
	<p>Arrows indicate top of product Transport and store in an upright position</p>
	<p>Fragile Do not throw or drop</p>

Text/symbol	Meaning
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

4. STARTING UP THE DEVICE

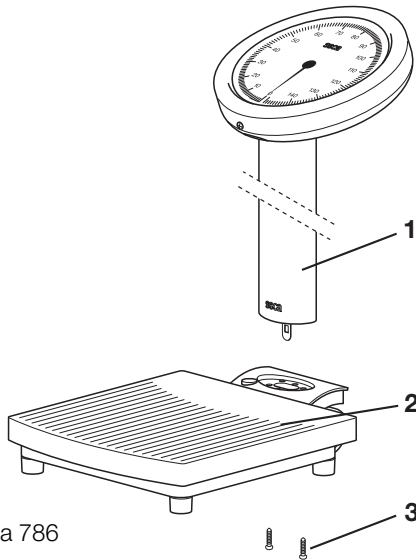
4.1 Scope of delivery


Example: seca 786

Item	Component	Pcs.
1	Weighing platform	1
2	Column with display	1
3	Screw	2

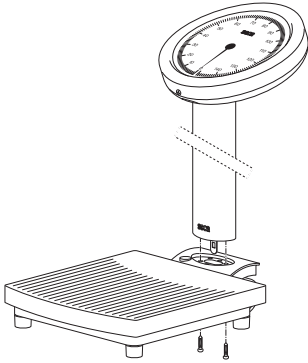4.2 Assembling the device

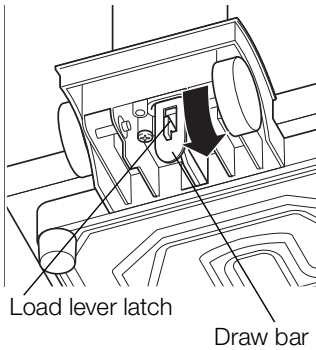
1. Turn the column so that the display is facing the weighing platform.
2. Guide the draw bar through the opening and position the column in the recess.
3. Screw the column to the weighing platform from underneath.

4. The scale can be carefully positioned on its side to tighten the screws.
5. Hook the draw bar over the load lever latch.
6. Then set the scale upright again.

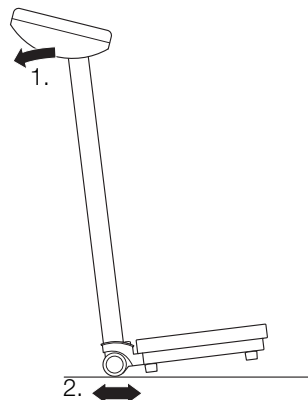4.3 Transporting the device

For mobile use, the scale can be tilted back slightly and then moved about on the rollers.

4.4 Setting up and aligning the scale

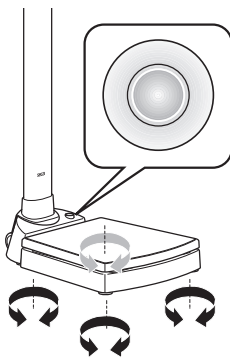
CAUTION!

Faulty measurement as a result of devices being set up incorrectly


Soft floors, e.g. wooden boards, give under a patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor for precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.

1. Place the device on a firm, level surface.
2. Align the device by turning the foot screws (only **seca 755/765**).

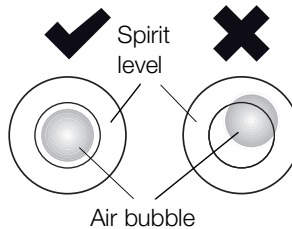The air bubble in the spirit level must be located in the exact center of the circle (only **seca 756**).

3. Ensure that only the feet of the device are in contact with the ground.

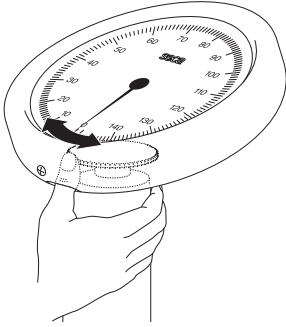
5. WEIGHING

CAUTION! **Personal injury**

Perform a function check before every use as described in the section “Function check” on page 38.

5.1 Adjusting the display

To obtain meaningful measured values you must check before every use that the pointer of the scale reads “0”.

1. Ensure that the scale has no load.
2. Check the current position of the pointer.
3. Move the adjusting wheel so that the pointer reads “0”.
 - Turn clockwise: Pointer moves counter-clockwise
 - Turn counter-clockwise: Pointer moves clockwise

5.2 Performing weighing

CAUTION! **Patients can be injured if they fall**

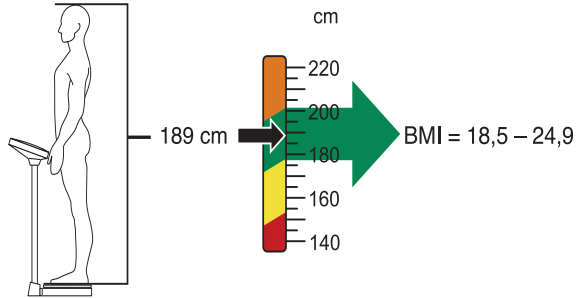
People with restricted mobility may fall when trying to step on or off the scales.

- ▶ Support people with restricted mobility when they step on and off the scales.
 - ▶ Only use this scale for patients who are able to stand on their own when being weighed.
1. Make sure that the pointer reads “0”.
 2. Ask the patient to step onto the scale and keep still.
 3. Read off the measured result.
 4. Ask the patient to step off the scale.

5.3 Reading off Body Mass Index (seca 755/756 only)

You need the patient's height.

1. Ask the patient to step onto the scale and keep still.
2. Read off the colored value at the associated patient height.

< 18,5	Underweight
18,5 - 24,9	Normal
25 - 30	Slightly to moderately overweight
> 30	Severely overweight

6. HYGIENIC REPROCESSING

NOTICE!

Damage to device

Inappropriate detergents and disinfectants may damage the sensitive surfaces of the device.

- ▶ Use only disinfectants free of chlorine and alcohol which are explicitly suitable for acrylic sheet and other sensitive surfaces (active ingredient: quaternary ammonium compounds, for example).
- ▶ Do not use white spirit or petroleum spirit.
- ▶ Only use disinfectants suitable for sensitive surfaces. Suitable disinfectants are available from specialist dealers.

6.1 Cleaning

- ▶ Use a soft cloth dampened with mild soapsuds to clean the surfaces of the device.

6.2 Desinfection

1. Check that your disinfectant is suitable for sensitive surfaces and acrylic sheet.
2. Follow the instructions on the disinfectant.
3. Disinfect the device at regular intervals using a soft cloth dampened with a suitable disinfectant.

Interval	Component
Prior to every measurement with direct skin contact	Weighing platform
After every measurement with direct skin contact	Weighing platform
If required	Scale ring and viewing window

6.3 Sterilisation

This device may not be sterilised.

7. FUNCTION CHECK

- ▶ Perform functional check prior to each operation.

A complete functional check includes:

- Visual inspection for mechanical damage
- Visual inspection and function check of the display elements
- Function check of all controls shown in the section “Overview”

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the section “Troubleshooting”.

**CAUTION!**
Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the section "Troubleshooting", you may not use the device.

- ▶ Have the device repaired by seca service or an authorized service partner.
- ▶ You should also follow the instructions in the section "Maintenance" on page 39.

8. TROUBLESHOOTING

Malfunction	Cause/Remedy
Pointer does not move under load.	The device is not ready for use - Check whether the draw bar is hooked in
Scale does not display a plausible weight value.	<ul style="list-style-type: none"> • Before weighing pointer does not read "0" - Adjust display • Force shunt: scale is e.g. touching a wall or patient is supporting his/her weight against the wall. - Move scale away from wall - Ask patient to stand on the scale unaided

9. MAINTENANCE

9.1 Information on servicing (seca 755/786)

The product must be set up carefully and serviced regularly. Depending on how frequently the scale is used, we recommended servicing at intervals of 3 to 5 years.

ATTENTION!**Incorrect measurements as a result of poor servicing**

- ▶ Please only have servicing and repairs performed by an authorized service partner.
- ▶ You can find service partners in your area at www.seca.com or by sending an e-mail to service@seca.com.

9.2 Information on verification (seca 756)

We recommend having your device serviced prior to verification.

NOTICE!

Incorrect measurements as a result of poor servicing

- ▶ Please only have servicing and repairs performed by an authorized service partner.
- ▶ You can find service partners in your area at www.seca.com or by sending an e-mail to service@seca.com.

Have an authorized technician perform verification according to national legal regulations.

Verification is necessary whenever one or more verification seals are damaged.

10. TECHNICAL DATA

10.1 General technical data

General technical data	
Dimensions seca 755/756 <ul style="list-style-type: none">• Depth• Width• Height	628 mm 360 mm 1000 mm
Net weight seca 755/756	approx. 18 kg
Dimensions seca 786 <ul style="list-style-type: none">• Depth• Width• Height	515 mm 292 mm 900 mm
Net weight seca 786	approx. 6,5 kg
Temperature range <ul style="list-style-type: none">• Operation• Storage• Transport	+10 °C to +40 °C (50 °F to 104 °F) -10 °C to +65 °C (14 °F to 149 °F) -10 °C to +65 °C (14 °F to 149 °F)
Air pressure <ul style="list-style-type: none">• Operation• Storage• Transport	700 - 1060 hPa 700 - 1060 hPa 700 - 1060 hPa

General technical data	
Humidity <ul style="list-style-type: none"> • Operation • Storage • Transport 	30 % - 80 % no condensation 0 % - 95 % no condensation 0 % - 95 % no condensation
Medical device in accordance with Regulation (EU) 2017/745	Class I with measuring function

10.2 Technical data, weighing

seca 756	
Calibration in accordance with Directive 2014/31/EU	Class IIII
Maximum load	160 kg
Minimum load	5 kg
Graduations	0.5 kg
Zero adjustment range	0 to 25 kg
Accuracy on initial calibration: <ul style="list-style-type: none"> • 5 kg to 25 kg • 25 kg to 100 kg • 100 kg to 160 kg 	±0.25 kg ±0.5 kg ±0.75 kg

seca 755	
Maximum load	160 kg / 350 lbs / 25 sts
Graduations	0.5 kg / 1 lbs / 1 lbs
Zero adjustment range	-20 kg to +45 kg / -44 lbs to +99 lbs / -3 sts to 7 sts
Accuracy: <ul style="list-style-type: none"> • to 80 kg • 80 kg to 160 kg • to 175 lbs • 175 lbs to 350 lbs • to 12,5 sts • 12,5 sts to 25 sts 	±0.5 kg ±0.6 % ±1.1 lbs ±0.6 % ±0.1 sts ±0.6 %

seca 786	
Maximum load	150 kg
Graduations	0.5 kg
Zero adjustment range	-17 kg to +45 kg

seca 786	
Accuracy:	
• to 60 kg	±0.5 kg
• 60 kg to 150 kg	±0.8 %

11. OPTIONAL ACCESSORIES

Accessory	Article number
Stadiometer seca 224	224 17 14 004

12. DISPOSAL

This device consists of high-quality raw materials which can be recycled. Dispose of the device in accordance with the local laws and regulations applicable in your country.

13. WARRANTY

We offer a two-year warranty from the date of delivery for defects attributable to faulty material or poor workmanship. This excludes all moveable parts such as (rechargeable) batteries, cables, power supply units, etc. Defects which are covered by the warranty shall be rectified free of charge for customers on production of the sales receipt. No further claims can be accepted. The costs of shipment in both directions shall be borne by the customer where the device is not located at the customer's premises. In the event of any damage during shipment warranty claims can only be asserted where the complete original packaging was used for shipment and the device was secured inside in the same manner as in the original packaging. You should therefore keep all packaging.

The warranty shall become null and void where the device is opened by persons not expressly authorised to do so by seca.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

14. DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

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